

# Pathway for use of PROMOGRAN™ Matrix and PROMOGRAN PRISMA™ Matrix in practice (adapted from)

Early adoption of PROMOGRAN™ Matrix and PROMOGRAN PRISMA™ Matrix is recommended at 4 weeks

Deliver Best Practice Care for 4 weeks as per national guidelines\*, such as offloading, pressure redistribution/relief, compression

How is the wound progressing against best practice guidelines and the original objectives?

PROGRESSING

NOT PROGRESSING

Continue treatment

Was this to be expected

YES

NO

e.g. end-of-life patient  
Continue treatment

Have the first principles of care been adhered to?

- First principles of care:**
- Assessment and referral
  - Good patient–clinician relationship
  - Dressing selection/suitability
  - Framework for practice

YES

NO

Are there signs of inflammation?  
Consider referral

Review & apply first principles of care! Consider referral

YES

NO

What is the risk of infection?

What is the risk of infection?

HIGH

LOW

HIGH

LOW

PROMOGRAN PRISMA™ Matrix

PROMOGRAN™ Matrix

PROMOGRAN PRISMA™ Matrix

Re-assess the diagnosis, has anything changed?

Review after 4 weeks, is the wound progressing?

YES

NO

Continue treatment

Re-assess and review the diagnosis, has anything changed?

\*e.g. NICE (2014) Pressure ulcers: prevention and management; NICE (2015) Diabetic foot problems: prevention and management; NICE (2017) Clinical Knowledge Summaries: Leg Ulcer - Venous; SIGN (2010) Management of chronic venous leg ulcers.